


Thermometer Reading Instructions

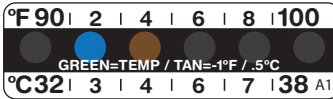

To Read the thermometer:

Green = Exact Temp

If you do not see green, then look for blue and tan colors. Blue add 1°F, Tan subtract 1°F. So, if the dot under 92° is blue and dot under 94° is tan, the true temp is 93°F. If reading in degrees Celsius you add/subtract .5°C. There are actual instructions about reading the thermometer printed on each unit (in the black section just above the Celsius scale).

Storage Recommendations:

Store out of direct sunlight.

Shelf Life:

Two years from date of manufacture.

Distributed by:

TransMed Company
1595 Peachtree Pkwy
Cumming, GA 30041
800-644-4145
www.transmedco.com
sales@transmedco.com